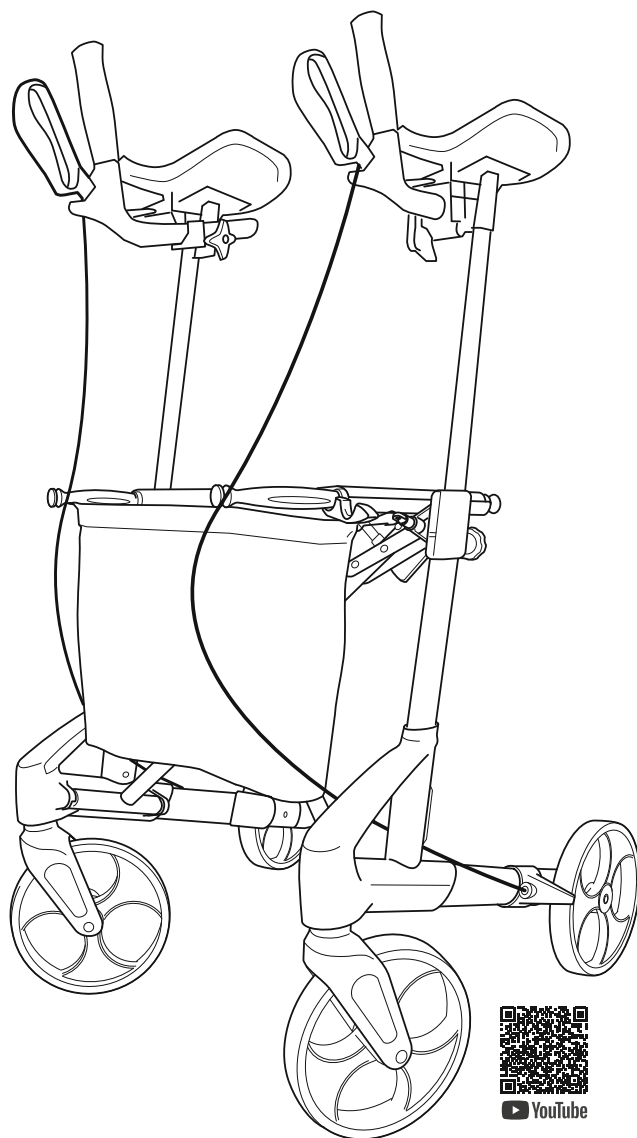
TOPRO

Instructions for use TOPRO Troja Walker²

topromobility.com

made in Norway

EN

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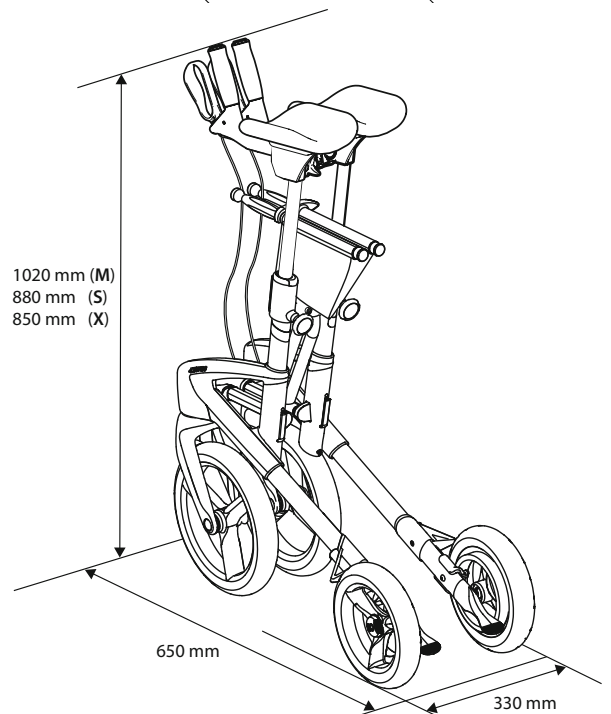
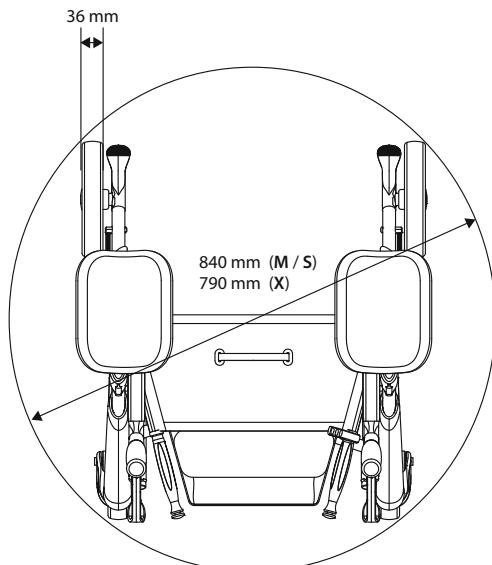
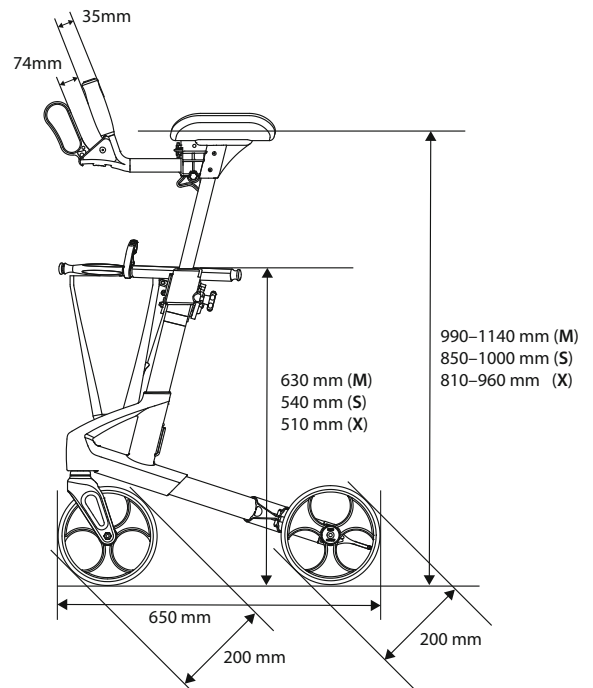
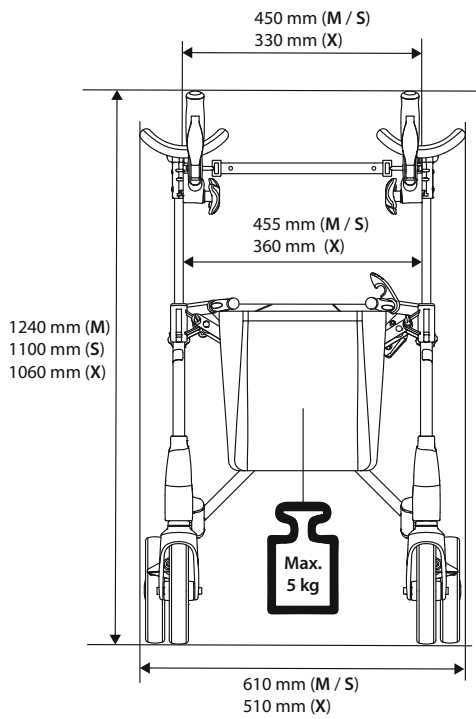
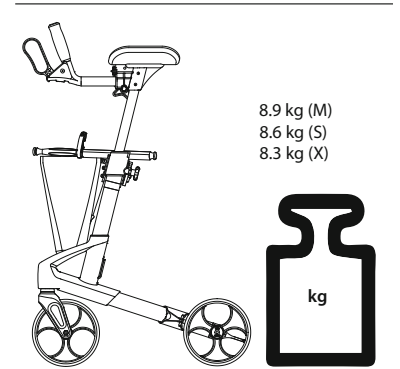
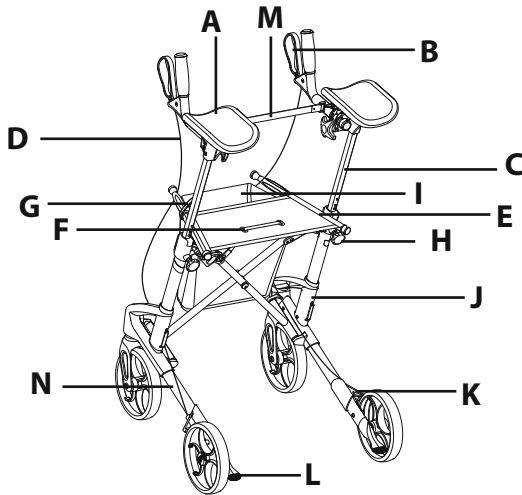
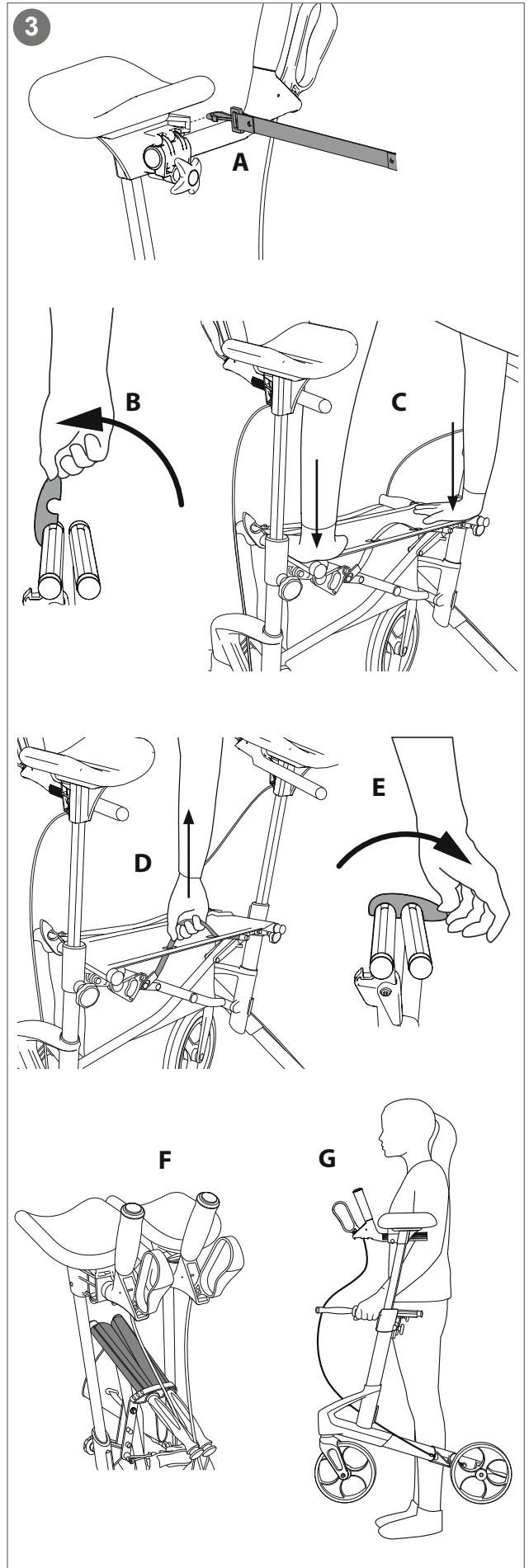
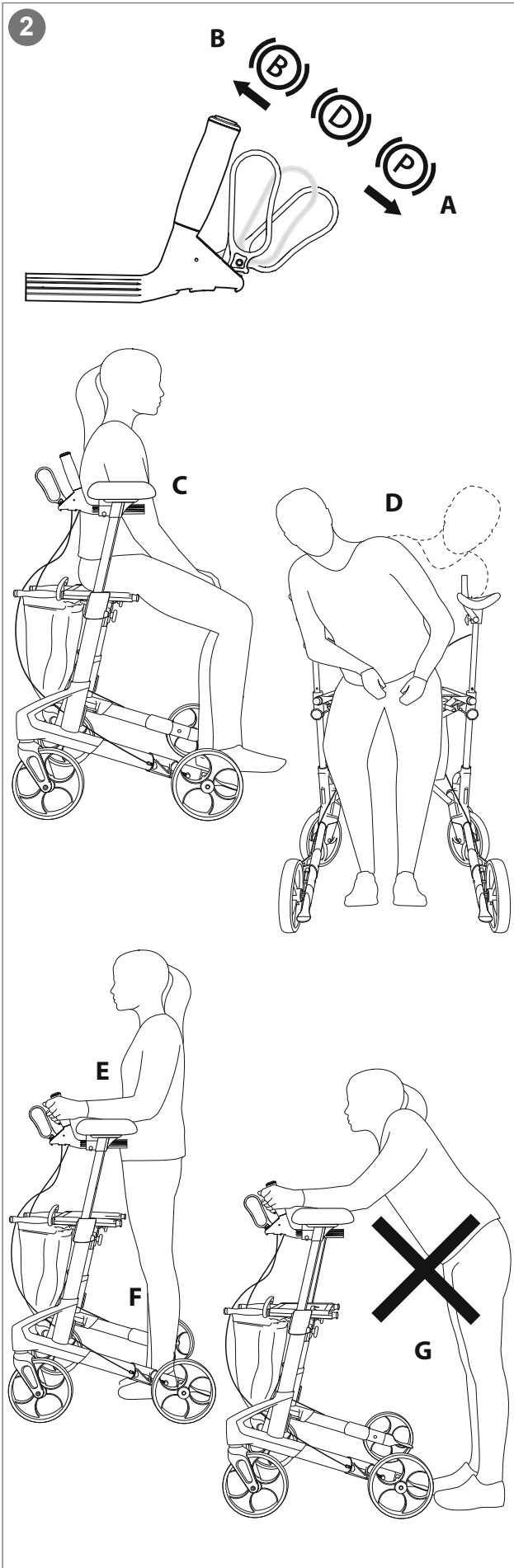
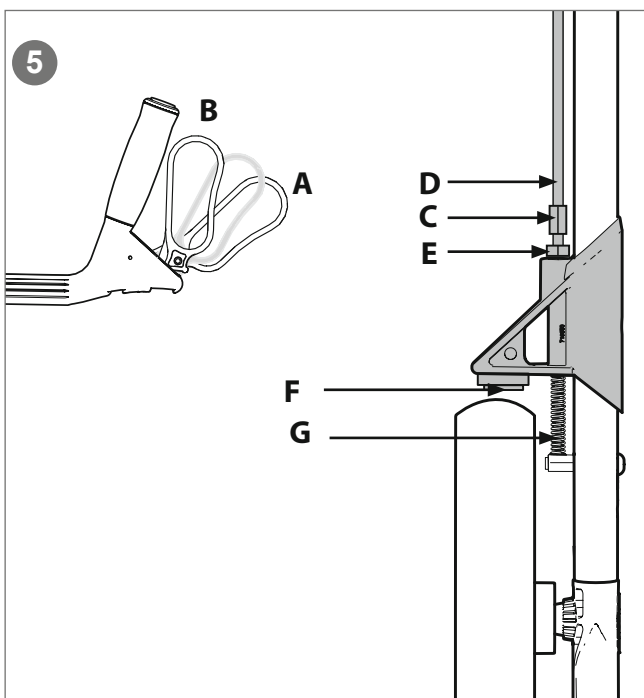
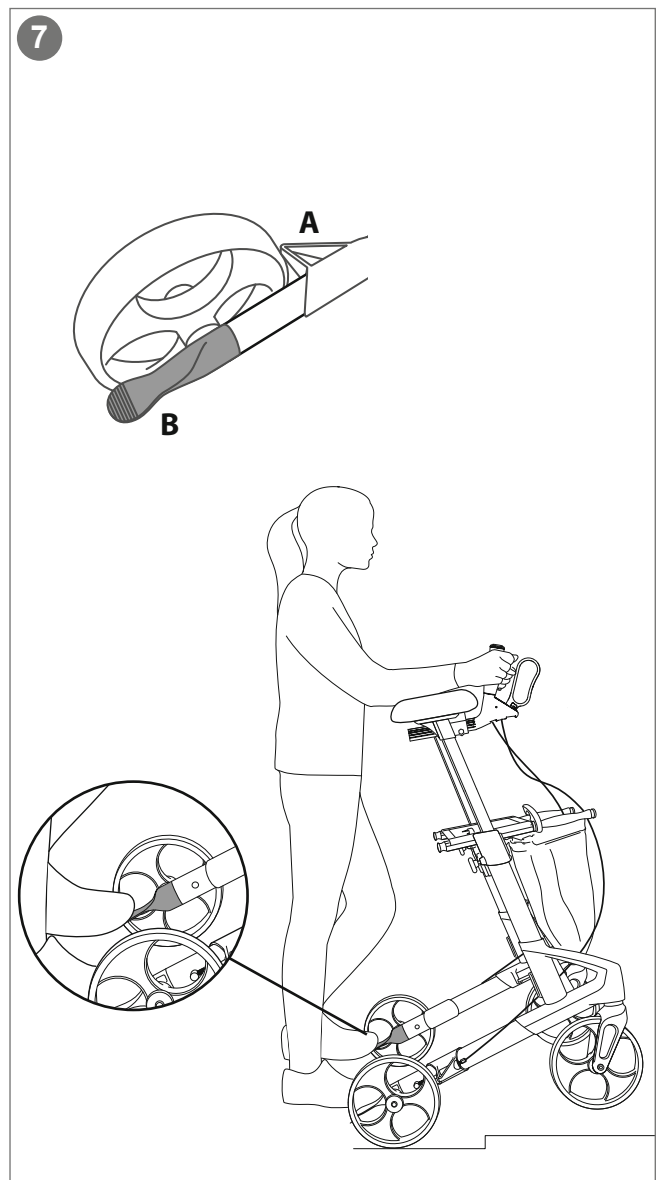
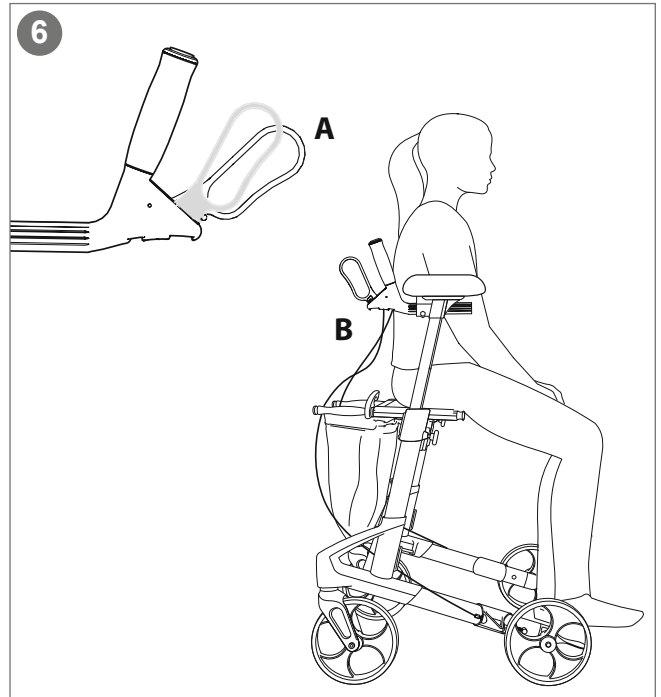
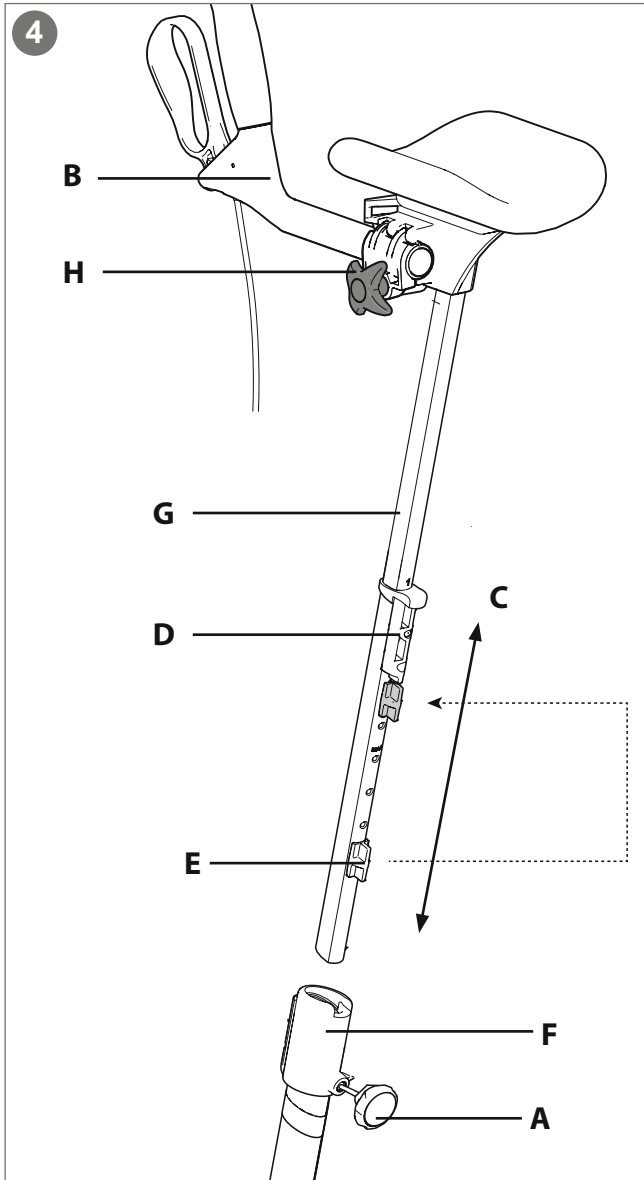
TOPRO Troja Walker² (M)

Model no. 815430

1 TOPRO Industri AS
Rambekkvegen 5, NO-2816 Gjøvik, Norway

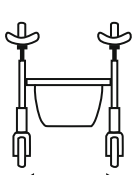
2021-12-14

Max 135kg

Max 650mm

Max 610mm

Max 1240mm

0000295477812

7054328154306

www.topromobility.com

9

10

11

13

2

12

14

3

4

5

6

7

8

TOPRO Troja Walker² (M)

Model no. 815430

1 TOPRO Industri AS
Rambekkvegen 5, NO-2816 Gjøvik, Norway

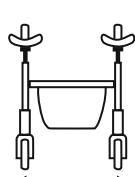
2021-12-14

Max 135kg


Max 650mm

Max 610mm

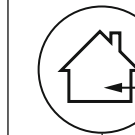Max 1240mm

0000295477812

7054328154306

www.topromobility.com

9

10

11

13

12

2

14

3

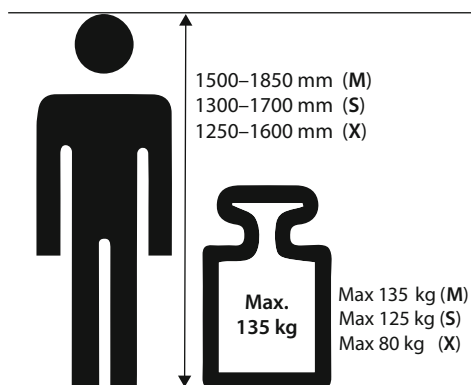
4

5

6

7

8

Congratulations with your choice of a new TOPRO walker

Every TOPRO forearm walker is developed, designed and manufactured in Norway. The product conforms to the Medical Device Regulation (EU) 2017/745. It is tested and approved according to EN ISO 11199-3:2005. On receipt of the goods, please check your product. If you have any queries, please contact your mobility shop or dealer immediately. If you have problems reading this user manual, an electronic version is available on topromobility.com.

Intended use

The TOPRO Troja Walker² shall give support to users with reduced balance and / or reduced walking ability. It is designed to be pushed, not pulled. The integrated seat provides an opportunity for the user to rest if needed. The intended user is an adult. Height and weight restrictions apply. *Contraindications: The product is not suitable for persons with low strength in arms, with very poor balance, or with considerable cognitive disabilities. The walker is designed and approved for indoor use on plain foundation.*

To get to know the walker and learn how to use it correctly, training by a skilled person is advised.

0 The grey numbered circles in this user manual refers to the numbered information graphic and illustrations inside the cover

1 Get to know your TOPRO product

- | | |
|--|------------------------------------|
| A Forearm support | H Lock wheel for the handle |
| B Brake lever | I Basket |
| C Handle tube | J Side frame |
| D Brake cable | K Brake block / Edge guard |
| E Seat bars / carrying handle | L Tilting function |
| F Seat with strap for folding mechanism | M Supporting cord |
| G Locking clamp | N Product label |

2 Safety

When folding and unfolding, be aware so you do not hurt your fingers.

Be aware not to put your fingers in open holes or between stationary parts.

Make sure that the lock wheel for height adjustment for the handles is locked tight.

When stationary or parked, make sure the parking brakes on both sides are activated (A).

Do not use the walker on stairs. Be careful when passing thresholds etc.

Check the brakes before use every time, and that the walker locks in unfolded position.

Do not use the device to transport heavy loads or persons.

Be aware that some parts of the walker might feel hot or cold when exposed to extreme temperatures.

Do not modify the product as this might put your safety at risk and the guarantee will become void.

Before sitting on the seat (C), make sure the parking brakes on both sides are activated (A).

To avoid the risk of falling, do not lean or bend sideways or over the back of the walker whilst in the seated position (D). Do not tilt the walker too much sideways, to avoid the risk of falling. For the same reason, bags etc. must never be hung on the handlebars.

To give the right support and safety, walk straight and make sure the walker is kept as close to your body as possible (E). Walk between the rear wheels (F), not behind them (G).

Be aware that the walker may roll faster than you when walking downhill, which may put you in risk of falling. Activate the driving brakes to adjust the speed (B).

Be careful when wearing fluttering skirts or trousers as they could get caught in the wheels, which could lead to falling. Wear sturdy footwear to avoid injuries.

Do not use the walker on slippery surfaces.

Keep the walker away from fire and hot items.

Do not climb on the walker.

3 Unfolding / Folding

Unfolding: Loosen the strap between the forearm supports (A) (not applicable for the walker in size X). Release the lock clamp positioned at the seat bars (B). Press the seat bars down and outwards (C). The side frames will slide apart. Press down the seat bars. You will hear a clicking sound. The walker is now locked in its open position. Fasten the strap (A).

Folding: Loosen the strap between the forearm supports (A). Pull the strap on the seat (D) and press the side frames together. Lock the clamp (E) over the seat bars.

Carrying the walker: The walker must not be carried by the seat strap or the brake wires. Fold the walker together as shown on the illustration (F). Carry the walker by the seat bars when the walker is folded (G).

4 Forearm supports / Handles / Memory function

Height adjusting: Unscrew the lock wheel (A) (approx. 2 turns). Adjust the forearm support tube to a suitable height then pull it slightly up and down (C) until you hear a clicking sound. Tighten the lock wheel (A). Adjust both tubes to the same height.

Recommended height: Stand between the rear wheels. Relax your shoulders and hold your elbows in a 90-degree angle. The level of the forearm supports should be fixed at the level of your elbows. A professional should help you with the adjustment of the forearm supports and the handles, to help ensure correct posture.

Adjusting the handles: Unscrew the lock wheel (H). You can now pull the handle (B) backwards or push it forward. It is also possible to adjust the angle of the handle. After adjusting the handle to the desired position, tighten the lock wheel (H) again. Be careful when unscrewing / tightening the lock wheel to avoid hurting your fingers.

Memory function: Decide on the height of the forearm support tube. Count visible holes at the back of the handle tubes (G). Unscrew the lock wheel (A). Pull the tube (G) out of the frame. The fastening piece (D) will now follow the handle tube. Move the fastening piece (D) so that the number of holes above it equals to the previously counted holes. Fit the red memory chip (E) on the tube in the first available hole under the fastening piece (D). Push the tube (G) and the fastening piece (D) downwards and into the side frame (F). Screw the lock wheel (A) halfway in. Pull the tube (G) up until it stops, and then tighten the lock wheel (A) properly. From now on, when the handle tubes are in the lowest position, they will stop in the pre-set position when pulled up.

5 Brakes

Make sure the brakes are working before every trip. Each brake lever (A) works on each of the rear wheels.

Do not drive or push the walker while the parking brakes are activated!

Parking brakes: Push the brake levers forward (A) to activate the parking brakes. Be aware that the lever has some resistance before it stops and locks the rear wheel. Pull the brake levers backwards to release the parking brakes.

Driving brakes: Pull the brake levers towards you (B) to reduce speed. *Do not use continuously while the walker is in motion.*

Adjusting / checking: Loosen the nut (C). Tighten the brake cable (D) with the adjustment screw (E). Adjust both brake blocks (F) so the distance to the wheel surface is approximately 1 mm and then tighten the nut (C). Make sure the brakes are not adjusted too tight and pressing hard into the rubber of the wheels when the parking brake is activated.

Maintaining: Clean the brake block (F) and the brake spring (G) regularly. The wheels and brakes can be replaced if they are worn out or damaged.

6 Seat

The walker has a safe and convenient net seat, through which you have a view to the ground while walking. It can easily be wiped with a cloth if needed. The parking brakes must be activated before using the seat. Push both brake levers forward until they lock (A). Both rear wheels are now locked. Sit with your back towards the basket (B).

7 Edge guard / Tilt function

Edge guard: The brake blocks (A), which are in front of the rear wheels prevent the rear wheels from catching on corners, door frames etc.

Tilt function: To use the tilt function use the front of the foot and press down on the tilt pedal (B), which is beside the rear wheel. This helps to tilt the walker over thresholds, edges etc. Please note: Be careful when passing thresholds etc.

Basket

The basket holds up to 5 kilos. It can be removed and washed gently at 60 °C.

Transport

The walker should be transported in the folded and locked position (see section *Unfolding / Folding*). Ensure that the lock clamp is on, so that the walker cannot unfold unintentionally. Be careful when loading the walker into a vehicle, giving attention to securing it.

Storage

The walker should be stored in an upright position. If stored outdoors, then under cover to protect it. Do not leave your walker unattended. Do not put heavy items on top of the walker during storage.

Cleaning

The walker can be cleaned using a damp cloth. In case of mud and grime accumulation, a cloth with a mild washing detergent can be used followed by rinsing with lukewarm water. Do not use high pressure cleaners and avoid flushing directly against ball bearings (in wheels and front forks). Do not use abrasives or steam wash. Dry the walker using a soft cloth.

Disinfection

Disinfection must only be carried out by authorized personnel. Please contact your local dealer or mobility aid centre. During disinfection sufficient protective equipment must be worn, and the instructions of the disinfectant manufacturer must be followed. The surface of the walker should be cleaned with a soft cloth or sponge and an approved chemical disinfectant containing 70–80 % ethanol. We advise against disinfectant that contains chlorine or phenol. Do not use heat disinfection. *The manufacturer cannot be held responsible for any damage or injuries that might occur using harmful disinfectant/measures or disinfection carried out by unauthorized personnel.*

Inspection / Maintenance / Re-use

It is recommended that inspection and maintenance are carried out regularly (frequency is determined by how and how often the walker is used). Check the following items: Frame, screws, tubes, handles, forearm supports, brakes, brake parts, wheels, seat and accessories. This also applies when the walker is to be prepared for re-use. Please see detailed instructions in this user manual about maintenance of brakes and cleaning / disinfection. The recommended maintenance is not a requirement, and no preventive maintenance is required provided that the walker is used as intended according to this user manual.

Material / Recycling

The walker is made of plastic-coated aluminium tubes, plastic couplings and material made of plastic and polyester. There are ball bearings in all wheels and forks. Most parts can be recycled. Dispose the walker and its packaging according to the applicable regulations in your country. For information, please contact your public authorities.

Warranty

TOPRO Troja Walker² is guaranteed free for faults and defects for 7 years. Damage caused by incorrect use or parts that are exposed to natural wear and tear (e.g. brake block, brake cable, wheel, basket, seat and grip) are exempt from the 7-year warranty. For repairs during the warranty period, please contact your local mobility shop or dealer. The warranty is void if unauthorised spare parts or accessories have been or are being used on the product. The expected life span of the product is estimated to be 10 years, if it is used in the correct manner according to this user manual and its safety and maintenance instructions.

8 Product label

- 1 - Manufacturer
- 2 - Consult Instructions for use (blue icon on the new label)
- 3 - Model name
- 4 - Model number
- 5 - Date of manufacture
- 6 - Serial number
- 7 - Global Trade Item Number
- 8 - GS1 DataMatrix
- 9 - Maximum user weight
- 10 - Maximum length of walker
- 11 - Maximum width / height of walker
- 12 - Intended for indoor use
- 13 - Medical Device
- 14 - CE marking (This product conforms to Medical Device Regulation (EU) 2017/745)

Accessories

	TOPRO Item No.
Back support	814765
Tray (NB: not for size X)	814728
Anti-slip mat for tray (NB: not for size X)	814059
Crutch holder	815358
Bottle holder	814043
Stabilizer bar (size M / S)	814729
Stabilizer bar (size X)	814730
One-handed brake	814026
Guiding handle for the visually impaired	814014
Rear bag with zipper (NB: not for size X)	814046
Name tag	814024
Roll Resistance System (RRS), pair	814032
Basket for oxygen bottle (only compressed, not liquid)	814009

Accessories mounted on the walker may influence stability, we therefore advise you to use them with care. The accessories can be ordered separately to enable a TOPRO Troja Walker² to be customised to individual needs. Contact your mobility shop, dealer or TOPRO for an updated overview of the accessories or visit our homepage topromobility.com.

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TOPRO Limited
TOPRO Mobility AB