# D-LITE

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## Table of contents

Preface .....	10
Declaration of conformity .....	10
Service life .....	10
Disposal .....	10
Identification plate .....	10
Reuse.....	10
Warranty .....	10
Parts of the product .....	11
Intended purpose .....	12
Indication .....	12
Scope of delivery .....	12
Safety instructions .....	12
Folding   Transport.....	13
Adjusting the armrests .....	13
Adjusting the footrests.....	13
Backrest with folding mechanism .....	13
Height adjustment of the push handles .....	13
Assembly of the leg support .....	14
Assembly and disassembly of the 24" rear wheels .....	14
Parking brake .....	14
Operating the tilting aid .....	14
Use in a vehicle .....	14
Cleaning   Maintenance .....	14
Technical data.....	15

## Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

## Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

## Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

## Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

## Identification plate

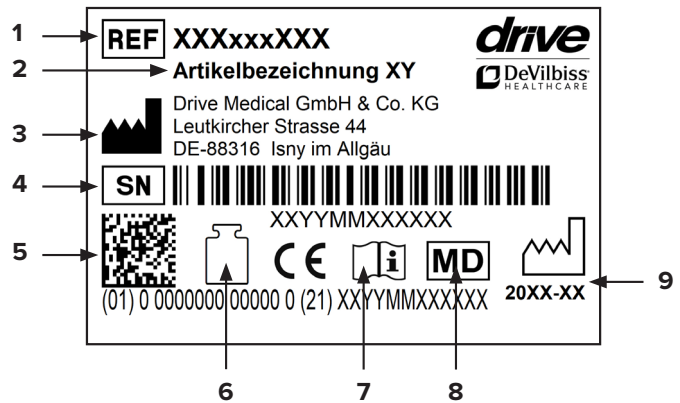1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

## Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at [www.drivedevilbiss.de](http://www.drivedevilbiss.de)

## Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

### EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes, handles and armrests

## Parts of the product

## D-LITE

- 1. Push handle
- 2. Back cover
- 3. Armrest
- 4. Leg support

- 5. Front wheel
- 6. Parking brake
- 7. Rear wheel
- 8. Tilt aid

- 9. Full-floating axle
- 10. Folding mechanism
- 11. Height adjustment of the push handles

## Intended purpose

Transport wheelchairs are intended for short-term use, indoors and outdoors (care facilities, hotels, airports, stations etc.) with an assistant/carer. They are used where there is significant to fully developed impairment of mobility/walking, with structural and/or functional damage of the lower extremities (including amputation, consequences of injury, musculoskeletal/neuromusculoskeletal movement disorders) and inability to propel a wheelchair independently (e.g. with limited strength and grip or lack of coordination of the arms/hands)

## Indication

### INDICATION:

Significant to full impairment of mobility/walking in the case of structural and/or functional damage to the lower extremities (including amputation, after-effects of injuries, musculoskeletal/neuromusculoskeletal movement disorders) and inability to independently use a wheelchair (e.g. in the case of limited strength and grip function or a lack of coordination ability of the arms/hands).

### CONTRAINDICATION:

Only for D-Lite 12.5“: Push wheelchairs are suitable for outdoor use only to a limited degree. Due to the rear wheels which are smaller in comparison to push-rim wheelchairs, it is more difficult to get over obstacles such as curbs.

## Scope of delivery

1 x	Transport wheelchair
1 x	Seat cover
1 x	Back cover
1 x	Instructions for use

## Safety instructions

To ensure proper and safe use, please be sure to observe the following information:

**Use the transport wheelchair only on even and firm surfaces.**

### Stability and balance

Your transport wheelchair is designed such that it ensures the stability you need during normal daily activities. Each of your movements in the transport wheelchair has an effect on the center of gravity which in the worst case can lead to tipping of the transport wheelchair. To increase your safety, you must use the safety belt.

### Reaching an object beyond the chair

The balance limits for reaching an object were determined through studies on a representative group of wheelchair users. Only the arms should be extended out beyond the wheelchair seat. The body and the head should remain within the limits of the seat.

### Do not lean forward

Do not lean your chest over the armrest. To reach an object in front of you, you must lean forward and bend downwards. For this reason, you must use the steering wheels as an aid (by facing them forwards) in order to attain stability and balance. Accurate orientation of the wheels is important for your safety.

### Do not lean back

Position the transport wheelchair as close as possible to the desired object so that you can easily reach it with outstretched arms while you remain seated in the chair in a normal position. Do not lean back too far over the backrest in any case since you can otherwise cause the chair to tip over.

### Lateral transfer

This transfer can be performed alone, provided you have appropriate mobility and muscle tone.

Position the transport wheelchair as close as possible to the place where you wish to sit such that the steering wheels are pointing forwards. Secure the parking brakes of the rear wheels.

Use a transfer board, if possible.

### Forward transfer

If you are able to stand up, you can transfer yourself forwards to another seat. If this movement is difficult for you, we recommend obtaining assistance from an attendant or caregiver to ensure a transfer that is as easy and safe as possible.

Secure both brakes and ensure that both steering wheels are pointing forwards. In doing so, be aware that you should not under any circumstances support yourself on the brakes during the transfer.

Swivel the footrests out of the way. Then grasp the top of the armrests, support yourself on your arms in order to easily get up and the attendant can assist you in completing the transfer.

Never stand on the footrests and never use the footrests as assistance when getting up.

### Tilting, overcoming obstacles with an attendant

#### Getting up onto the sidewalk

The attendant positions the transport wheelchair facing forward in front of the sidewalk. He/she then tilts the transport wheelchair backwards towards him-/herself until the steering wheels reach up onto the sidewalk; he/she pushes the transport wheelchair forward until the

rear wheels are at the edge of the curb and pushes the transport wheelchair further until the rear wheels roll over the edge of the curb.

### Getting down off the sidewalk

The attendant positions the transport wheelchair facing forward with the steering wheels at the edge of the curb and tilts it backwards towards him-/herself. Then he/she carefully pushes the transport wheelchair forwards until the rear wheels touch the street, then he/she gradually lowers the steering wheels to the ground.

### Getting over stairs

The transport wheelchair should not be carried when in use by a patient.

To navigate stairs, the user must get out of the transport wheelchair and use the stairs, for example with the aid of an attendant. The transport wheelchair can then be carried up or down the stairs by the attendant.

- Do not attempt to lift the transport wheelchair by grasping any removable parts (for example, the armrests, leg supports or footrests).
- Avoid using an escalator.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.
- Protect the product from direct sunlight. Product parts can become hot and cause skin injuries if touched.
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

## Folding | Transport

The transport wheelchair was developed with a simple folding mechanism. When folding up the wheelchair, ensure that the leg supports are removed or the foot plates are folded up. Remove the seat and back cushions and also the rear wheels, if necessary. Grasp the front and rear edge of the seat surface in the center and pull them upwards. The wheelchair folds up by itself.

## Adjusting the armrests

The angle and the front height of the armrests can be adjusted to the user. To change the angle, press the button as shown in the photo and at the same time, move the front end of the armrest up or down. Once the desired angle has been reached, release the button to lock it (**pic. 1**).

## Adjusting the footrests

Use the enclosed Allen wrench to loosen the screw at the footrest and adjust the length as desired by shifting up or down (**pic. 2**).

## Backrest with folding mechanism

The transport wheelchair has a mechanism for folding down the upper half of the backrest to decrease its height and thus allow the transport wheelchair to be stowed in a space-saving manner (**pic. 3**).

### Attention:

Do not pull on the black lever since this can lead to damage and/or injuries.

## Height adjustment of the push handles

To adjust the height of the push handles, loosen the T-handle and pull the handle to the desired height (**pic. 4**). Tighten the T-handle once again!

## Assembly of the leg support

The leg supports can be swiveled inwards and outwards and can be removed if desired.

To unlock the leg support, release the lever (see arrow) and swivel the leg support inwards or outwards as needed (**pic. 5**).

To remove the leg support, swivel it about 45° outwards and pull it upwards. To replace the leg supports, insert them at an angle of about 45° and then swivel them inwards until they click into place.

### Attention:

Always make sure that the leg support is correctly inserted and clicked into place before use.

## Assembly and disassembly of the 24" rear wheels

The 24" rear wheels of the transport wheelchair can be easily removed for transport or storage.

To do this, hold the frame of the transport wheelchair with one hand while holding the center of the wheel with the other hand. Press the pin in the middle inwards and at the same time, pull the hub away from the wheel.

To reattach the wheels, perform these steps in reverse order (**pic. 6**).

The 12.5" wheels can be removed only with a tool.

### Attention:

Before use, check that the wheel is securely mounted by pulling on the wheel and ensuring that it does not move laterally.

## Parking brake

The transport wheelchair comes with two parking brakes which may only be used when the chair is standing still. To operate the parking brakes, press the brake handle down as shown in the photo (**pic. 7**).



To release the parking brakes, pull the brake handles back up.

Brake for attendant only in the case of the 12.5" version (**pic. 8**):

## Operating the tilting aid

To maneuver around curbstones and over thresholds step firmly (with the left or right foot) on the tilt aid (**pic. 9**) while simultaneously pushing down the handles. This ensures that lifting the front of the wheelchair is easier. Ensure that you are standing securely.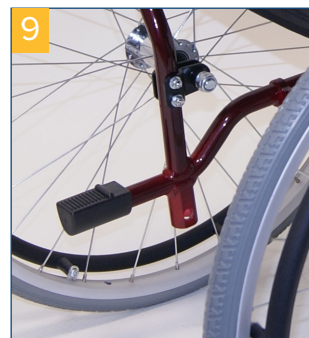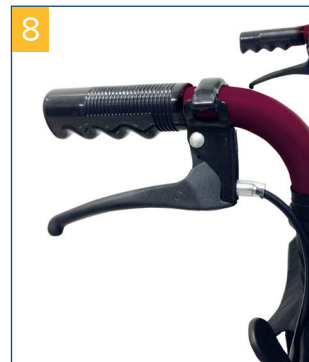

## Use in a vehicle

The wheelchair user must sit in a regular vehicle seat and the transport wheelchair must be securely stowed. The transport wheelchair was not designed for use as a seat in a motor vehicle; failure to observe the above instructions can lead to injuries.

## Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.
- Check the condition of the tires regularly. They are "fail-safe", meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage, however.

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Drive Medical GmbH & Co. KG